



MODERN DENTAL SLEEP SOLUTIONS. OVENTUS™ SLEEP APPLIANCES.

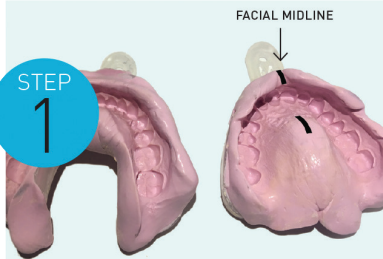

Benchmark impressions with upper facial
midline marked

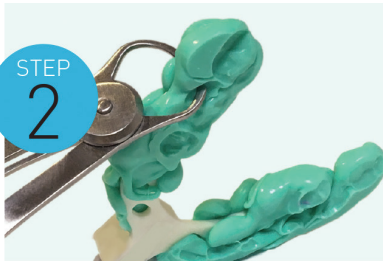

Calipers measuring the bite

IMPRESSION AND BITE RECOMMENDATIONS

TAKE UPPER AND LOWER IMPRESSIONS

Using standard dental techniques, record the patient's upper and lower dental impressions. Mark facial midline with permanent marker on the upper impression tray.

- > Use a fast set medium/heavy body PVS impression material. (Alginate impression material will degrade during transportation and is not suitable).
- > Ensure there are no drag marks and that all teeth in the arches are recorded.
- > To achieve adequate retention, the lower arch impression should extend up to 5mm beyond the gingival margin.

Impressions and bite registration should be decontaminated according to normal practice prior to shipping.

RECORD PROTRUSIVE BITE

Using a George Gauge or other bite registration tool of your choice, record the patient's bite at your preferred percentage of protrusion (commonly 50-60%). The bite record is used by Oventus to set the level of protrusion and titration range of the oral device.

- > Please use standard bite registration material (PVS or putty).
- > Ensure the vertical distance between the upper and lower arch is a minimum of 5mm universally. This is critical to incorporate the Oventus Airway Technology in the device design.
- > Check the vertical dimension at the anterior and posterior of bite record with calipers, measuring the deepest point of the cusp.

Although we are able to accept digital scans of the full arch upper and lower impressions, we are unable to accept digital scans of the open/protruded bite. Please send the physical bite to the Lab.

For more info, call 877-711-8778 or visit moderndentalusa.com