

MODERN DENTAL SLEEP SOLUTIONS. OVENTUS™ SLEEP APPLIANCES.

CARING FOR YOUR O₂VENT™ DEVICE



Your O₂Vent device is made of quality biocompatible materials to provide a comfortable and effective treatment. To maintain the lifetime of your device, we recommend you follow the cleaning regime.

DAILY CLEANING

After each use of your O₂Vent device, it is recommended that you follow these cleaning instructions:

- Remove the device from your mouth and rinse in lukewarm water.
- Clean the device with a soft clean toothbrush (not the same toothbrush you use to brush your teeth as toothpaste can damage the device).
- Rinse well with lukewarm water.
- Shake as much of the water as possible from the device, especially from the airways.
- Dry your device after each cleaning with a clean tissue or towel and place it back in the storage box.
- Ensure you clean your teeth before you insert your device.

Twice a week use an effervescent antibacterial cleaner for a more thorough cleaning. Please dry your device thoroughly and place it back in the storage box following cleaning.

For more info, call 877-711-8778 or visit moderndentalusa.com